



eFilm™ Scan™ Version Log

eFilm Medical Inc.
500 University Avenue
Suite 300
Toronto, Ontario, Canada
M5G 1V7
Tel +1.416.204.9664
www.efilm.ca

Bugs Fixed in eFilm™ Scan™ 1.0.1

1. The rotation of an image in the QA dialog is now preserved when the image is imported into eFilm Workstation.
2. The support for TWAIN compliant digitizers has been improved. In addition, for TWAIN scanners that do not automatically detect the size of the image you are scanning, the ability to set the image width and height has been added to the Scan dialog box.
3. The main eFilm Scan window is now resizable.
4. The responsiveness and speed when drawing segment boxes, moving thumbnails, and window levelling the images have been improved.
5. The thumbnail order is now preserved if the thumbnails have been saved and then restored during a future eFilm Scan session.
6. The Accession number field on the Import dialog now accepts both numeric and alphabetic characters and will automatically be filled with a NULL character if the field has been left blank.